

eyeCard[®], Set of 2

No. 510-776



Please read these instructions of use carefully before using the product!

Safety information

- Please read the safety information about using the product safely that is contained in these instructions carefully.
- Always keep these instructions at hand.
- Please check that the product is not damaged.
- Keep children away from packaging materials.

Intended use

- Only use the product in accordance with the description contained in these instructions of use.
- The product is not a toy. Keep children away from the product.
- The product has only been designed to enlarge details close-up and as a reading aid.
- Do not use the product as a replacement for your glasses when handling vehicles or when you're on the road.

Remark:

The product is not a vision aid. Using this product does not mean that you should not consult your doctor if necessary.

How to use the Eye Card reading magnifier

The Eye Card reading magnifier enables you to comfortably magnify the writing on credit cards and the "small print" on documents.

• Place the Eye Card reading magnifier on your nose as shown.

The retention clip may be easily bent to make the magnifier fit better. When you have finished using it, you can comfortably put the Eye Card reading magnifier in your wallet or purse as you would a credit card.

Storage

- Keep the product in a clean and dry place in its original packaging.
- Protect the product from dirt, dust, heat, moisture and water.



Care and cleaning

- Clean the product's surface with a commercially available solvent-free cleaning or care cloth for vision aids. Use a little washing-up liquid if necessary.
- Do not use any chemical cleaning additives.
- Do not use any materials (e.g. abrasive cloths) that could scratch the surface.

Product features

Magnification factor: 2.5x

Disposal

• Please dispose of the product in accordance with the locally applicable regulations.